



American Breast Care Products Care Instruction Sheet

Thank you for your purchase of American Breast Care products. With a little care, they will provide lasting comfort. Please follow the care instructions below.

Breast Form Care Instructions:

- Clean your breast form daily in lukewarm water with mild soap and dry off with a soft towel. Chlorinated or salt water will not harm your ABC breast form.
- Avoid stretching, twisting, rubbing or using excessive force when handling your ABC breast form.
- Store your ABC breast form nipple side down in the travel case provided.
- Avoid bringing your ABC breast form in direct contact with perfume, hot surfaces, and hot water or abrasive cleaners.
- Avoid sharp, pointed objects near your ABC breast form. Pins, fingernails, pet claws, scissors, etc. can all cause permanent damage.

Bra Care Instructions:

- Hand wash your ABC bra in warm water with a mild detergent. Do not use fabric softener.
- Line dry.

If you have any questions at all about the care of your ABC products, please let us know!

Comfort Compression

4918 Temple Ave Suite F Evansville IN 47715 8123033831

Website: comfortcompression.com

Business Hours: Mon- Thu 9-4; Fri 8-12

Your fitter's name: _____